




How does the BiWaze Clear System work?

BiWaze® Clear System provides oscillating lung expansion (OLE) therapy to help treat and prevent atelectasis, remove retained secretions from deep in the lungs and reduce work of breathing. BiWaze Clear expands and clears the airways through a combination of airway clearance therapies in just 10 minutes. Alternating therapies of lung expansion and high frequency oscillation combined with nebulization* helps calm inflamed airways and thin mucus.

BiWaze® Clear Therapy = 10 minutes*

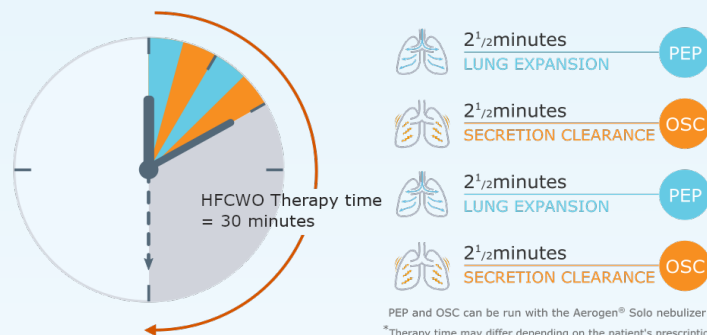

Differentiators

- Expiratory flow bias during oscillation aids in more efficient and effortless secretion clearance
- Superior aerosol deposition to the lungs¹ compared to other OLE and IPV systems
- Dual air-path control separates inhaled and exhaled air, enabling a closed-circuit design
- Integrated closed-circuit design features an Aerogen® Solo nebulizer* and built-in Aerogen controller
- Store up to 10 therapy profiles with advanced programming to customize each therapy phase
- Dynamic touchscreen control and an intuitive user interface for an enhanced user experience
- A lithium-ion battery is included with each system and provides power for up to 10 therapies
- Light weight system is only 9 lbs, including the lithium-ion battery



SUPERIOR AEROSOL DELIVERY

In vitro study found BiWaze Clear delivers a greater deposition of aerosol to the lungs compared to the Volara System. The closed, dual lumen breathing circuit with the Aerogen nebulizer* has a profound impact on aerosol delivery.¹

5x more aerosol during PEP therapy¹

3x more aerosol during OSC therapy¹

2x less fugitive aerosol¹

1. DiBlasi, R, et al. (2023). BiWaze Clear Aerosol Comparison White Paper. Seattle Childrens Hospital and Research Institute. <https://abmrc.com/wp-content/uploads/2023/07/BiWaze-Clear-Aerosol-Evaluation-White-Paper.pdf>

HCP CS CODES

E0469 = Lung expansion airway clearance, continuous high frequency oscillation, and nebulization device

A7021 = Monthly supplies and accessories for lung expansion airway clearance, continuous high frequency oscillation, and nebulization device (e.g., handset, nebulizer kit, biofilter)

GOVERNMENT PROGRAM

Distributor: First Nation Group

Email: Orders@FirstNationGroup.com

Phone: 1-855-221-5332

* FDA CLEARED FOR SALINE SOLUTION ONLY WITH NEBULIZER

RESOURCES



website



overview



training

TARGET PATIENT POPULATION

BiWaze Clear can be used throughout the continuum of care, hospital to home for pediatric and adult patients.

PATIENT CONDITIONS

- Bronchiectasis
- COPD
- Cystic Fibrosis
- Amyotrophic Lateral Sclerosis
- Muscular Dystrophy
- Spinal Muscular Atrophy
- Spinal Cord Injury
- Quadriplegia

PATIENT SYMPTOMS

- Diminished lung ventilation
- Shallow breathing/hypoventilation
- Inadequate airflows
- Mucus plugging
- Poor or absent cough
- Pain associated with chest wall injury or incisions
- Impaired diaphragm

Rx Only

BiWaze® and ABM Respiratory Care® are registered trademarks of ABMRC, LLC.

©2024 ABMRC, LLC. ALL RIGHTS RESERVED. PRN-1585539264-381 Rev 4.0 OCT-2024 ENG US

For further information about this product or a service, visit our webpage: www.abmrc.com or call customer support: 1-877-ABM-RC01 (877-226-7201)

abm
RESPIRATORY CARE